FR

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EN

Warning and safety instructions

Observe the instructions for use and all safety instructions for this product. Non-compliance can reduce the effectiveness of the medical facemask.

Wash / disinfect your hands thoroughly before putting on the mask and after removing it. The mask needs to cover your mouth and nose and must not be worn under your chin.

Avoid touching the mask and your face with your hands while using the mask.

Dispose of the facemasks after use in accordance with the local disposal regulations.

The protective mask is intended for single use only. A used or severely contaminated facemask must not be used again/continue to be used - it must be disposed of immediately. Observe the recommended or stipulated replacement intervals for the facemask according to health and safety directives or hygiene directives.

Specification

Medical face masks Type IIR DIN EN 14683:2019-10
 Shelf life (in their original packaging): When stored in the closed bag - 5 years.

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EN Manufacturer Information

Medical Face Masks Type IIR, DIN EN 14683:2019-10

Intended purpose: Protection for patients and other persons from infectious germs, to prevent the risk of spreading infections, particularly in pandemic situations. Type IIR masks provide elevated protection from liquid splashes.

Performance characteristics:

Bacterial filter capacity $\geq 98\%$,

Splash protection ≥ 16 kPa,

Breathing activity < 40 Pa/cm²,

Microbiological purity ≤ 30 KBE/g

CE as a Class I medical product in accordance with EU 2017/745

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EN

How to use

1. Wash your hands with soap and water for at least 20 seconds before putting on the medical facemask. Remove the mask from its packaging and hold it with both hands by the ear loops. Check for damage and contamination before use.
2. You can identify the top surface of the mask by the sewn-in flexible wire which is used to fit the mask to the bridge of the nose. Put on the mask with the white side on the inside so as to cover your nose and mouth. Place both the ear loops over your ears so that the mask sits firmly in place.
3. Fit the nose clip to the bridge of your nose from the centre outwards using both index fingers.
4. Pull the lower part of the mask over your chin. The mask should fit as snugly as possible on the bridge of your nose and on your cheeks. Avoid touching the mask when wearing it. The mask should be replaced every two hours or if it becomes damp. Single-use products must not be worn more than once and must only be worn by one person.
5. Hold the mask by both ear loops when removing it. Dispose of the mask directly into a waste bin with a lid. Then wash / disinfect your hands.

A detailed technical datasheet and comprehensive user instructions are available as a PDF in the download area on the TechniSat Homepage at www.technimask.de.

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