TechniSat

[DE] Elektronische Gebrauchsanweisung
[EN] Electronic user manual
[FR] Manuel d'instruction électronique
[PL] Elektroniczna instrukcja obsługi

[DE] **Medizinische Gesichtsmasken Typ IIR**
[EN] **Medical Face Masks Type IIR**
[FR] **Masques chirurgicaux de type IIR**
[PL] **Medyczne maski twarzowe typ IIR**

TechniSat

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EU-Declaration of Conformity

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[EN]

Instructions for use for medical devices
according to EU 2017/745 and EU 207/2012
Version 1.0 dated 13/11/2020

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Identification / UDI:
0000/2733 / 0000/2733

Typ:
Medical Face Masks
Type IIR DIN EN 14683:2019-10

EU Compliance Assessment:
Class 1 medical device according to EU 2017/745

Article numbers:
0010/2733 – Bag with 10 masks
0100/2733 – Bag with 10 masks
0050/2733 – Package with 5 bags
0500/2733 – Package with 5 bags
1010/2733 – Package with 100 bags

Note on packaging:
The designations 1002 and M2 R on the secondary packaging correspond.

Intended purpose:
Protection of patients and other persons from infectious germs for the avoidance of the risk of propagation of infections, particularly under pandemic conditions.

Performance characteristics:
Bacterial filter capacity $\geq 98\%$
Breathing activity $< 40 \text{ Pa/cm}^2$
Microbiological purity $\leq 30 \text{ KBE/g}$
Splash protection $\geq 16.0 \text{ kPA}$

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[EN]

Indications:

Staying in environments where measures are required or recommended to reduce the risk of infection for the wearer or other persons.

Contraindications:

There are no general contraindications to the use of medical face masks. Limitations may result from the geometry of the mask in relation to the shape of the user's head (tight fit of the mask and covering of mouth and nose, see "intended user" above) or health conditions in which a slight increase in breathing resistance cannot be tolerated. Please consult your doctor if you have any complaints.

Users:

Adolescents and adults from the general population, including patients and staff of health care facilities such as operating theatres. The requirement in the instructions for use that the mask should cover the user's nose and mouth and that the mask should be firmly attached to the ear loops excludes some users, such as babies.

Use:

Non-sterile and for single use
Used masks are considered to be highly contaminated
No cleaning / disinfection
Observe instructions for use,
warning & safety instructions

Incident reporting:

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

Storage conditions:

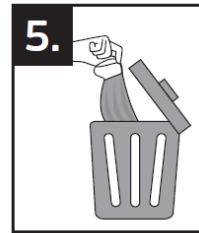
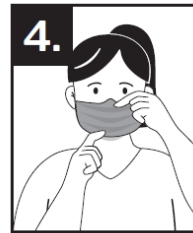
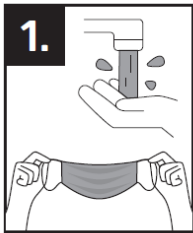
Temperature range -30°C ... +70°C
Relative humidity < 80%

Storage time:

5 years in original packaging under storage conditions

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How to use

1. Wash your hands with soap and water for at least 20 seconds before putting on the medical facemask. Remove the mask from its packaging and hold it with both hands by the ear loops. Check for damage and contamination before use.

2. You can identify the top surface of the mask by the sewn-in flexible wire which is used to fit the mask to the bridge of the nose. Put on the mask with the white side on the inside so as to cover your nose and mouth. Place both the ear loops over your ears so that the mask sits firmly in place.

3. Fit the nose clip to the bridge of your nose from the centre outwards using both index fingers.

4. Pull the lower part of the mask over your chin. The mask should fit as snugly as possible on the bridge of your nose and on your cheeks. Avoid touching the mask when wearing it. The mask should be replaced every two hours or if it becomes damp. Single-use products must not be worn more than once and must only be worn by one person.

5. Hold the mask by both ear loops when removing it. Dispose of the mask directly into a waste bin with a lid. Then wash / disinfect your hands.

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[EN]

Warning and safety instructions

- Observe the instructions for use and all safety instructions for this product.
- Non-compliance can reduce the effectiveness of the medical facemask.
- Wash / disinfect your hands thoroughly before putting on the mask and after removing it.
- Avoid touching the mask and your face with your hands while using the mask.
- The mask needs to cover your mouth and nose and must not be worn under your chin.
- Dispose of the facemasks after use in accordance with the local disposal regulations.
- The protective mask is intended for single use only.
- A used or severely contaminated facemask must not be used again/continue to be used - it must be disposed of immediately.
- Observe the recommended or stipulated replacement intervals for the facemask according to health and safety directives or hygiene directives.

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